## Panther Fusion® GI Assays



Stool processing guide using Cary-Blair transport media and Aptima® Multitest Specimen Collection Kit (PRD-03546)



Mix Cary Blair preserved specimen thoroughly to ensure homogeneity immediately prior to transfer into Aptima Multitest tube.



Using the swab from an Aptima Multitest Specimen Collection kit, completely submerge the soft tip into the Cary-Blair preserved stool specimen ensuring the pink shaft is not submerged.




Place the swab into the Aptima Multitest tube and swirl for 5 seconds to release material.


Replace the cap of

Aptima Multitest tube.



Break the shaft of the Aptima swab at the score line and leave the swab in the tube.

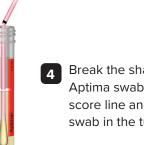

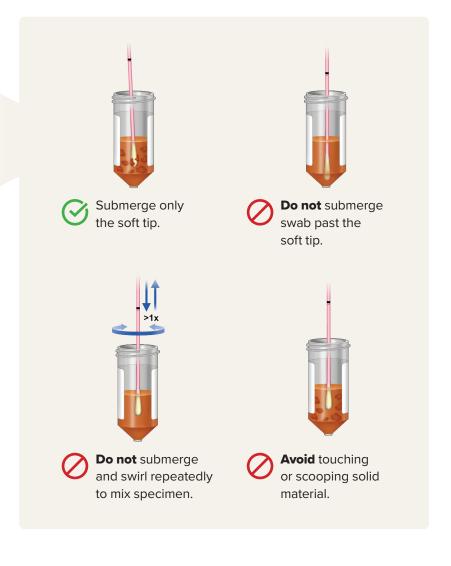

**( €** 2797 **EC REP** 

Hologic BV, Da Vincilaan 5, 1930 Zaventem, Belgium. NB number details where ever applicable.

Diagnostic Solutions | Hologic.co.uk | euinfo@hologic.com

Panther Fusion® GI Bacterial Assay Panther Fusion® GI Expanded Bacterial Assay